

How to use your BeatO CURV Glucometer



Please Enter Mobile Number to login Please Enter Mobile Number to login Enter phone number By proceeding to login, you are agreeing to our Terms & Conditions and Proacy Pulley STEP 2 Login by entering your phone number. You will receive an OTP and will be automatically redirected to the app home page
Open Beat0 to handle CP2102N USB to UART Bridge Controller? OutART Bridge Controller is connected Cancel OK STEP 5 A popup will appear. 'Always open' needs to be ticked. Press OK to move ahead.
STEP 8 Apply the blood sample after 'Insert blood' appears on screen
Image: second









Do's and Dont's

Please ensure your CURV is inserted front-facing





Ensure your strip is inserted front-facing





Make sure you apply enough blood at the edge of the strip. Do not apply blood on or below the strip.





Demo Videos





https://youtu.be/gWb8S5maVil